



Assembly Instructions

AM Series Head Rest

Part Number 30-0330

Contents of Head Rest for 1400 Seat Back

Adjustable Padded Head Support

Adjustable Support Rod

Head Support Rod


Three Hex Head Screws

Head Support Retaining Ring

Assembly Process

- 1) Place head support rod through head support retaining ring. Align three holes in retaining ring with three holes on the back of padded head support.
- 2) Using the three hex head screws to attach retaining ring to the back of the padded head support. You will need a 3 mm allen wrench for this step. See Picture

Retaining ring & Hex head-screws

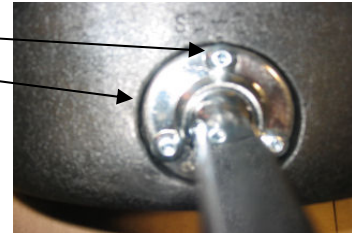

- 3) Remove black end cap from the end of head support rod by prying with screwdriver. See Picture

Black end cap



- 4) Unscrew adjustment knob completely from adjustable support rod. See picture

Adjustable knob on support rod



AM Series Head Rest Assembly Instructions

- 5) Insert head support rod through adjustable support rod and insert black end cap. See picture
Install adjustment knob by turning clockwise.



- 6) Remove push bar/handle from 1400 series seatback support rod by pulling upwards. In more difficult cases the use of a rubber mallet may be needed.
7) Slide head support assembly onto seat back tubes by sliding over and properly adjust.



- 8) Re-insert push bar/ handle onto 1400 series seat back support rod.

NOTE: BEFORE PLACING A CHILD ON THE AM SERIES AFTER INSTALLING THE HEAD REST SUPPORT ASSEMBLY CHECK THAT ALL SCREWS AND NUTS ARE TIGHT.