



Assembly Instructions

AM Series Knee Separator


Part Number 30-0310

Contents of Knee Separator

- Knee Separator Pad (Black)
- Knee Separator tube with cam lock lever
- Hex head threaded bolt
- Flat washer
- Allen wrench

Assembly Instructions

- 1) Insert the hex head bolt through the washer and then through the hole in the stem.
- 2) Align the hex head bolt with the end of the knee separator pad that has the threaded insert.
- 3) Turn the hex head bolt to the right (clockwise) until it is snug by hand.
- 4) Use the allen wrench to finish tightening the hex head bolt into the stem. See picture below.

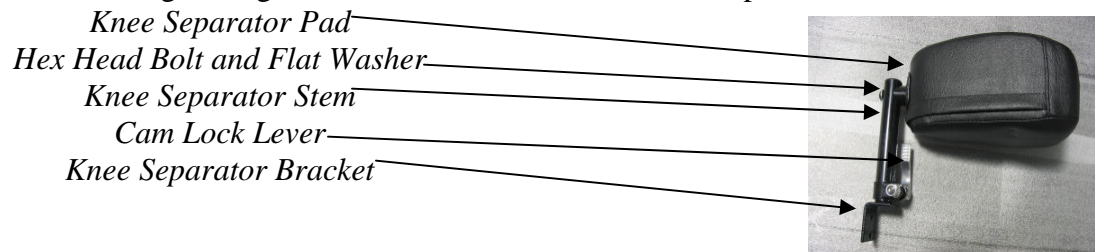


Installing the Knee Separator on the AM Series Therapeutic Trykes

- 1) Located under the front part of the seat post are two hex head bolts. See picture.

Hex Head Bolts

- 2) Using the allen wrench provided remove the two hex head bolts.
- 3) The bracket located at the end of the knee separator stem attaches to the seat post the two hex head bolts.
- 4) Align the holes of the knee separator bracket with the two holes located on the seat post.

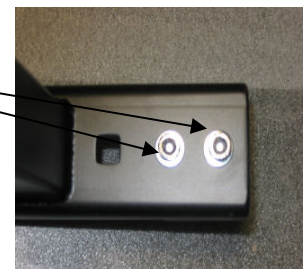

- 5) Install one hex head bolt at a time using only your hand to tighten. After both bolts are hand tight proceed by using the allen wrench to finish tightening the two bolts.

Adjustments

By pulling on the cam lock lever the knee separator will become loose allowing you to adjust the length or to remove the knee separator.

NOTE: BEFORE PLACING A CHILD ON THE AM SERIES AFTER INSTALLING THE KNEE SEPARATOR CHECK THAT ALL SCREWS AND NUTS ARE TIGHT.