



Assembly Instructions

Lumbar Pad

Part Number 30-0334

Fits # 30-0501 1400 Seatback only


Contents of Lumbar Pad

Lumbar Pad with mounting bracket and 2 knobs


Installation Instructions

- 1) Remove the push bar/ handle from the top of the 1400 seat back by pulling upwards. In more difficult cases a rubber mallet may be needed. If the push bar/ handle is not easy to remove, apply some lubricant into the seam between the push bar/ handle and the support tubes.
- 2) Remove existing padded back pad with lateral attachments by turning the two knobs located on the back counter clockwise (Left). Then pull upwards and off.

Turn knob counter clockwise to loosen

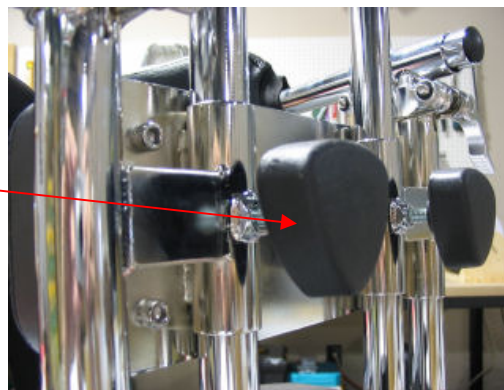

- 3) Loosen the two knobs located on the back of the lumbar pad mounting bracket before trying to install on support tube.

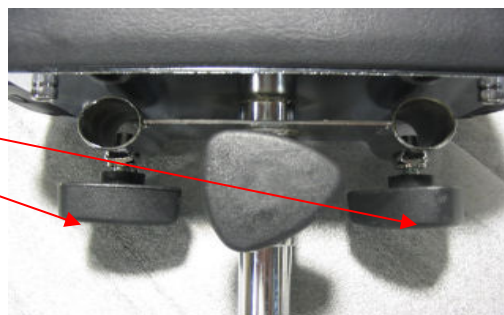

- 4) Slide the lumbar pad down the two support tubes to the desired location.
- 5) Re-install the padded back pad with lateral attachments and the push bar/ handle.

Lumbar Pad Adjustments

The lumbar pad has two adjustment features:

- 1) The position of the lumbar pad can go up and down the support tubes to the desired location.
- 2) The horizontal positioning of the lumbar pad is achieved by loosening the center knob.

NOTE: BEFORE PLACING A CHILD ON THE AMTRYKE AFTER INSTALLING THE LUMBAR PAD CHECK THAT ALL KNOBS ARE TIGHT.