



The therapeutic tricycle for children with disabilities



For questions or to place an order,
please contact:

AmTryke, LLC.
Phone: (888) - AMTRYKE ext. 14
Email: amtryke@ambucs.org
Visit our website: www.ambucs.org

Assembly Instructions

AM Series Push Bar

Part Number 30-0305

Contents

- Push bar with textured rubber handle
- Push bar bracket
- 9/16 Bolt
- 9/16 Nut
- 9/16 Flat washer

The purpose of the push bar is to provide the parent or therapist assistance in moving the child forward while the child is building strength and coordination to power the tryke by themselves. The push bar eliminates the need to bend over and push.

Note: The push bar is not meant to provide leverage for raising or lifting the AmTryke® therapeutic tricycle. If used in a manner other than specified above, National AMBUCS™, Inc. and AmTryke®, LLC are not responsible for breakage, damage, or injury.

Step 1: Attaching Push Bar Connector to Rear Deck

- Take silver U-bracket with bolt and nut out of bag.
- Remove nut and slide bolt from U-bracket.
- Carefully push U-bracket onto horizontal bar of rear deck that holds rear wheels. Slide bracket up next to vertical mainframe bar. Connector tube should face toward rear of tryke.
- Slide bolt back into U-bracket, making sure to match up the square part of bolt head with square hole. Screw on nut and tighten.

NOTE: BEFORE PLACING A CHILD ON THE AM SERIES AFTER INSTALLING THE PUSH BAR CHECK THAT ALL SCREWS AND NUTS ARE TIGHT.