



Assembly Instructions



Without Forward Extender Plate

AM-9S Forward Extender Plate Part Number 10-0109

Contents of AM-9S Forward Extender Plate

- Seat Extender Plate
- (2) Phillips head machine screws
- (2) Flat Washers
- (2) Locking Washers
- (2) Wing Nuts


Tools Needed: Phillips head screwdriver



With Forward Extender Plate

The AM-9 Forward Extender Plate allows the seat to be positioned closer to the front frame. The plate is mounted in between the seat and the seat frame bracket. Smaller children will be able to reach hand cranks and foot pedals.


1. Remove bucket seat from AM-9S if already installed. Keep the hardware to reattach the seat.

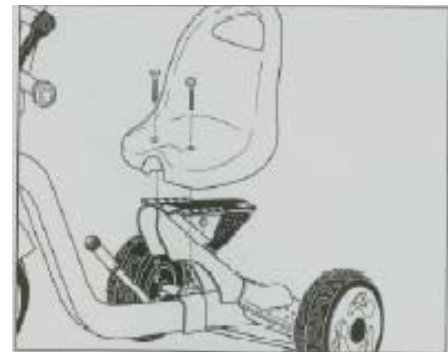


2. Place seat extender plate on top of seat frame bracket. Align the back 2 holes in the plate with holes in the frame bracket depending on the desired position of the seat. Note: Choosing frame bracket holes closer to the front of the tryke will position the seat closer to the front frame.

Revise Date: 12/4/08

Insert the 2 Phillips head screws,

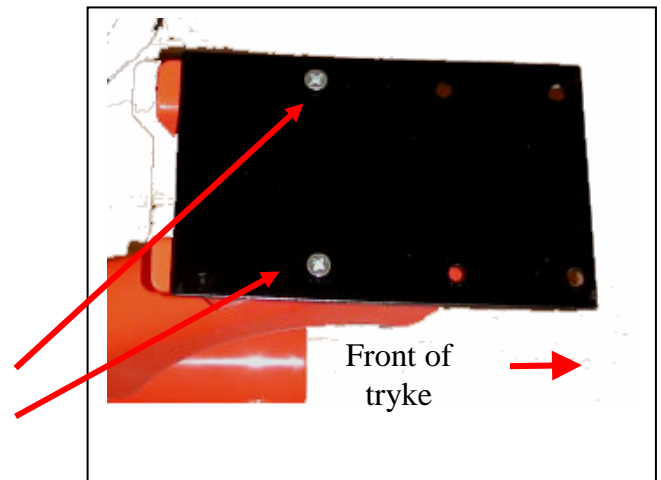

3. From the underside of the seat frame bracket install flat washer and locking washer onto screw, then install wing nut. Tighten securely. Repeat step for other screw.

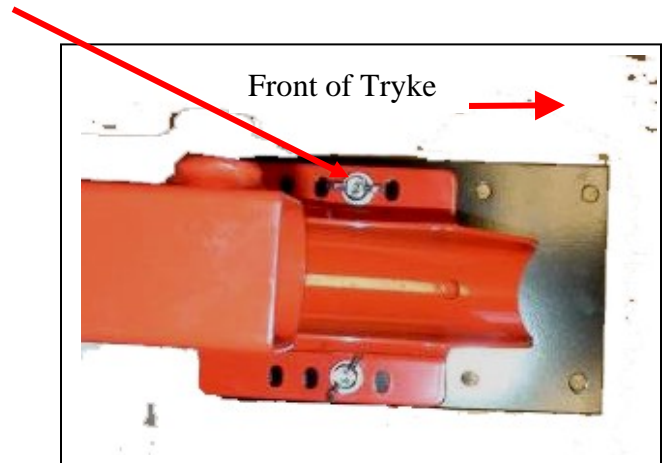

Use the original hardware that is provided with the AM-9S to mount the seat to the AM-9S frame. Install one screw through the seat and into the seat extension bracket hole using a washer and nut to secure the seat to the bracket. Repeat step for other screw.

NOTE: BEFORE PLACING A CHILD ON THE AM-9S AFTER INSTALLING THE FORWARD EXTENDER PLATE CHECK THAT ALL SCREWS AND NUTS ARE TIGHT.