



Assembly Instructions

Bucket Seat Installation Instructions

Part Number 30-0320

Contents of Bucket Seat

Grey plastic bucket seat with chest and lap straps
Four (4) Phillips machine screws
Two 10mm x 2" Allen head bolts
Two 10mm Hex nuts
Four flat washers
Black seat post
Black seat bracket


Tools Needed

Phillips Head Screw Driver
3 mm Allen Wrench
14 mm Wrench

NOTE: if this is an upgrade then remove current seating system before starting.

Installation Instructions for AM-12, AM-16 and Older Model AmTrykes

- 1) Insert black seat post into black seat bracket at a 90 degree angle, making sure the longer portion of the seat bracket is facing the same direction as the knee separator mount .

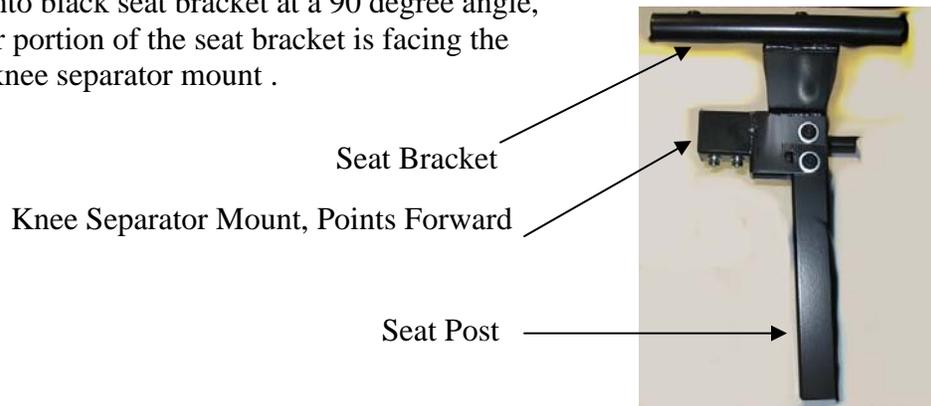

- 2) Slide one flat washer over each 10mm bolt. Insert 10mm bolt with washer on it through upper hole in seat bracket and through seat post until it comes out opposite side. Slide other 10mm bolt with washer on it through bottom hole (not slotted hole) and through seat post until it comes out the opposite side. Attach flat washer over each bolt sticking out. Install 10mm hex nuts on bolts by turning clockwise finger tight. Tighten using 3mm hex key and 14mm wrench.

Bucket Seat Installation Instructions Continued

- 3) Insert assembled seat post and seat bracket into frame of tryke making sure the side of the black seat post with the holes is on the same side as the tension knob in the tryke frame. Pull tension knob out until holes align, release knob and tighten.

Tension Knob



- 4) Set grey plastic seat on top of seat bracket. Insert one Phillips head screw through hole in grey plastic seat and the hole in the black seat bracket. Tighten one turn clockwise. Insert the remaining screws into the other three holes, tightening each screw one turn clockwise.
- 5) Once you have all four screws installed, tighten all four screws securely to finish the installation.




Installation Instructions for ProSeries 1412, 1416 and 1420 AmTrykes

- 1) Insert black seat post into black seat bracket at a 60 degree angle.

Seat Bracket

Knee Separator Mount, Points Forward

Seat Post

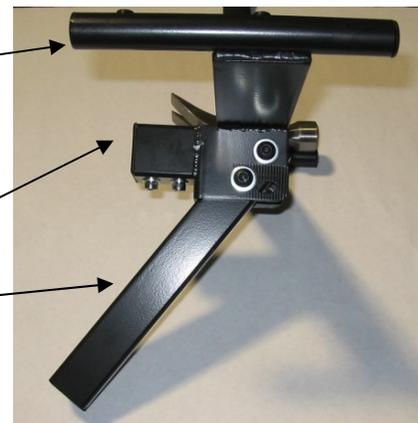

- 2) Slide one flat washer over each 10mm bolt. Insert 10mm bolt with washer on it through upper hole in seat bracket and through seat post until it comes out opposite side. Slide other 10mm bolt with washer on it through slotted and through seat post until it comes out the opposite side. Attach flat washer over each bolt sticking out. Install 10mm hex nuts on bolts by turning clockwise finger tight. Tighten using 3mm hex key and 14mm wrench.
- 3) Follow Steps 3, 4, and 5 from above instructions.

NOTE: BEFORE PLACING A CHILD ON THE TRYKE AFTER INSTALLING THE BUCKET SEAT BRACKET AND SEAT CHECK THAT ALL SCREWS AND NUTS ARE TIGHT. FAILURE TO SECURELY TIGHTEN ALL SCREWS COULD RESULT IN AN INJURY.