



## Assembly Instructions

### Hand Strap

#30-0100

Can be used on all trykes with removable hand grips

#### Contents

30" Strap with detachable plastic tab and hook and loop piece.

**Tools Needed:** 15mm wrench

#### Hand Strap Installation

- 1) Using the tryke's 15mm wrench, remove the desired side (right or left) hand grip. Loosen hand grip by turning the wrench in the direction towards the back of the tryke.
- 2) Remove the plastic tab from the hand strap.
- 3) Insert the threaded portion of the hand grip through the hole in the plastic tab. Make sure the hook fastener is away from the hand.
- 4) Replace the hand grip into the tryke and tighten firmly. Tighten hand grip by turning the wrench in the direction towards the front of the tryke.
- 5) Reattach the strap to the hook material on the plastic tab.

#### Hand Strap Usage

Place hand on the hand grip with fingers on top and thumb on the under side. Using either side of the strap, go around the wrist and then over the fingers until the strap is all used up. Finally use the end piece of hook and loop to secure strap. Make sure that the hook is not touching the skin.

When not in use, leave the plastic tab on the hand grip and remove and store the strap and end piece until needed.

Note: the 30" strap is very soft for comfort but it may wear out at the base where it attaches to the plastic tab. Trim off a short section of the strap.

The foam strap is manufactured by flame lamination so that it will withstand laundering.

