



## Assembly Instructions


### Contents of Seat Extension

- Seat Extension Plate
- (2) Phillips Head machine screws
- (2) Hex Nuts with nylock thread lock
- (2) Round flat washers

### Assembly Process

- 1) Remove seat from AM-9S if already installed.
- 2) Position seat extension plate over top of area where seat mounts.
- 3) Align long slotted holes with the two front mounting holes of the AM-9 seat frame. (See Picture)

First hole  
recommended

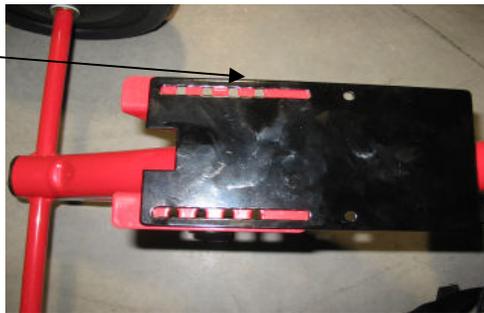

- 4) Insert one screw through seat extension plate and through AM-9S seat mounting frame. Repeat process for the other screw. (See Picture)

Phillips Head  
Machine Screw



- 5) Install washer and nylock nut onto screw from bottom. Tighten securely. Repeat step for other screw.
- 6) Position seat onto seat extension bracket, making sure holes in seat align with bracket.
- 7) Mounting the seat requires the use of the original hardware that is provided with the AM-9S to mount the seat to the AM-9S frame.
- 8) Install one screw through the seat and into the seat extension bracket hole using a washer and nut to secure the seat to the bracket. Repeat step for other screw.

**NOTE: BEFORE PLACING A CHILD ON THE AM-9S AFTER INSTALLING THE SEAT EXTENSION BRACKET CHECK THAT ALL SCREWS AND NUTS ARE TIGHT.**