



Assembly Instructions

AmTryke Snappy Seat #30-1500

This seat may be used on the Snappy Tricycle (#50-0150), AM-9 models (#50-0102) and AM-12 (#50-0210 using special mounting bracket #30-1503)



Carton Contents

- 1-Padded seat back
- 1-Padded head rest
- 1-Padded seat
- 2-Threaded seat knobs
- 1-Seat Belt
- 1-Metal seat bracket

Tools Required

- 1-5/32 Allen Wrench

- 1) Attach the padded head rest to the padded back by using four Allen screws and four flat washers as shown. Use a 5/32 Allen wrench to tighten.




- 2) Attach the metal seat bracket to the back side of the padded back using four Allen screws and four flat washers as shown. Use a 5/32 Allen wrench to tighten. Note: Two round metal rods point towards the front.



- 3) Slide the padded seat over the metal seat rails as shown.



- 4) Insert the two threaded knobs and tighten.

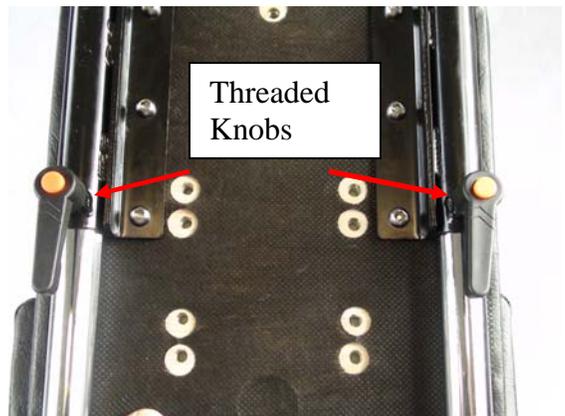

5) Attach the seat belt by using one lag bolt and one wing nut on each side as shown.




6) Attach seat belt center strap with one Allen screw and a flat washer to bottom of padded seat as shown.



7) Located on the bottom of the padded seat are eight pre-drilled and threaded holes that are used for mounting the Snappy seat to either the Snappy, AM-9 or AM-12.



Snappy Seat Parts

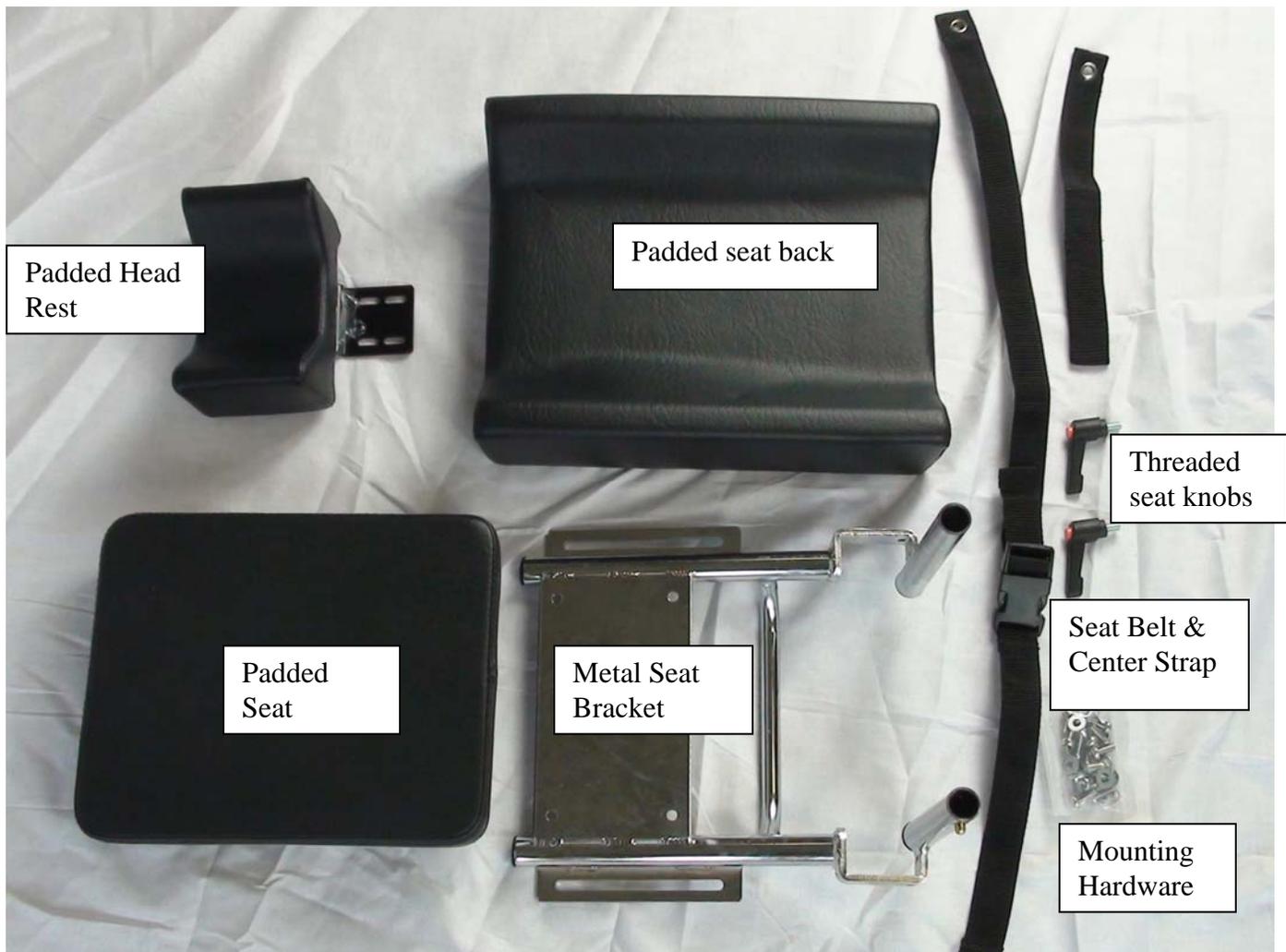

Additional Accessory Parts used on Snappy Seat

Small Laterals - #30-1501

Large Laterals - #30-1502

H-harness – #30-0336 - 11.5-inch

H-harness - #30-0337 – 8-inch

Adaptor bracket for mounting on AM-12 #30-1503