



The therapeutic tricycle for children with disabilities



For questions or to place an order,
please contact:

AmTryke, LLC.

Phone: (888) - AMTRYKE ext. 14

Email: amtryke@ambucs.org

Visit our website: www.ambucs.org

Assembly Instructions

#30-24-0005

Stem Riser for 1416, 1420 and 2700 series AmTrykes

Included:
Stem Riser
6mm Long Hex Key

- 1) Use 6mm long hex key to loosen 6mm bolt that is in the center of the steering stem.
- 2) Pull up on handlebar/stem and remove from front end of tryke.
- 3) Insert stem riser into front end of tryke to the maximum height line on stem riser.
- 4) Use 6mm long hex key and tighten stem riser securely.
- 5) Insert handlebar/stem into stem riser.
- 6) Use 6mm long hex key and tighten stem into stem riser.