



Assembly Instructions

AmTryke® Wrist Wrap

XS = 7 inches and smaller - # 30-21-0160xs

S = 8 inches - # 30-21-0160s

MED = 10 inches - # 30-21-0160med

LG = 11 inches - # 30-21-0160lg

XL = 12 inches - # 30-21-0160xl

Wrist Wrap Directions

1. Place hand on tryke hand grip/handle bar with thumb placement below grip and fingers above hand grip.
2. Place small T or L end of wrist wrap around the top side of the wrist and secure hook and loop.
3. Pull material snug over back of hand toward underside of wrist.
4. Secure hook and loop fastener of strap around wrist / back of hand.
5. Discomfort, constriction of blood flow or red marks should be avoided.

Measure from point **A** to **B** for correct size.

Xtra Small	Small	Medium	Large	Xtra Large
7"	8"	10"	11"	12"

