



The therapeutic tricycle for children with disabilities



For questions or to place an order,
please contact:

AmTryke, LLC.

Phone: (888) - AMTRYKE ext. 14

Email: amtryke@ambucs.org

Visit our website: www.ambucs.org

Assembly Instructions

50-FC-1416-3 & #50-FC-1420-3
Pro Series 1416 and 1420 Three Speed Models


The 3-speed ProSeries models are suitable for higher functioning children and adults. These models feature 3-speed internal hubs and coaster brakes. Key features are the 3 speed options and the ability to coast. This contrasts with the basic ProSeries models that teach pedaling and force full range of motion through the positive drive function (no coasting).

Riders for the new 3-speed coaster version must be able to make a complete pedal rotation on their own. The trykes are suitable for riders who need help with balance and stable sitting positions. Riders must have the cognitive level and ability to steer successfully, apply the coaster brake and/or the hand brake. This model is designed to allow the freedom for a rider to use a bike trail, tackle small terrain changes, and ride for extended periods with family and friends.


Tools Needed

5mm Allen Wrench

- 1) Insert shifter pin into rear hub.

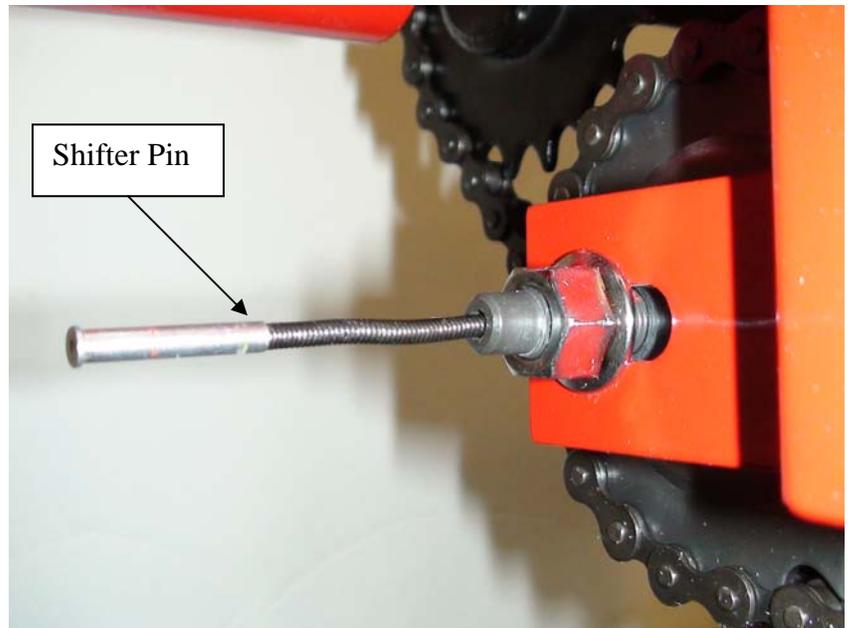

2) Slide shift mechanism over hub bolt, firmly hold in place while tightening the set screw with a 5mm allen wrench.

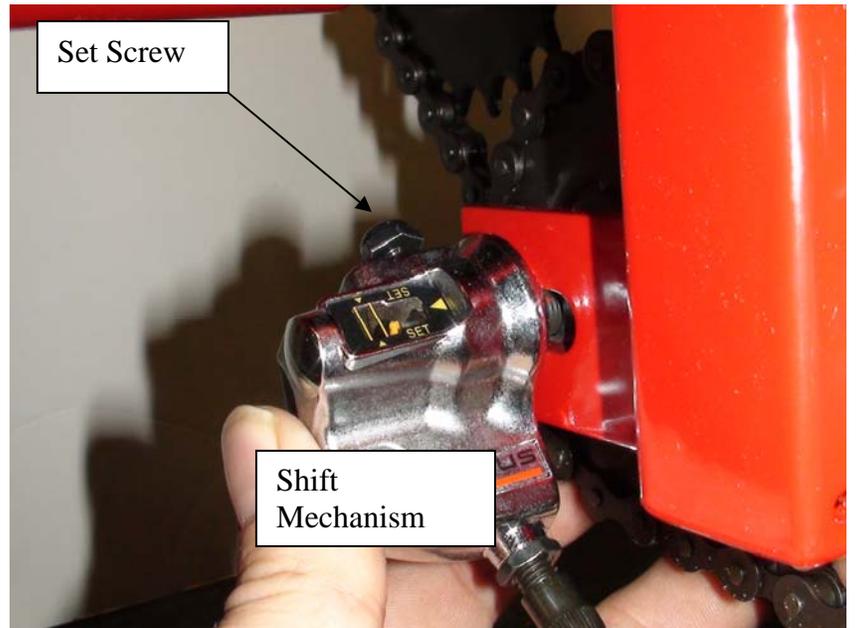

3) Install combination shift control and hand brake lever on handlebar. Use 5mm allen wrench to tighten.



4) The shift control can also be mounted optionally on the loading brake stem.



Combination Shift Control and Hand Brake Lever.



Shifter Pin



NOTE: The hand brake lever can be removed from the shift control and be mounted on the handlebar.