

Does your child need help keeping their feet on the pedals?

If so, use the included black hook and loop straps and the directions below to secure your child's feet to the pedals. Each child is different and placement of the foot on the footplate should be determined by the needs of your child. Use the instep and the heel strap in combination to assist your child.

Figure A. Instep Strap

Begin by attaching one end of the longer (30-inches) black strap to the Velcro® on the bottom of the footplate below the heel of the child's foot. Wrap the strap up and over the child's instep and continue the wrapping under the footplate making sure the strap attaches firmly to the Velcro® on the underside of the footplate. Cross over to the other side of the footplate firmly attaching the strap to the Velcro®. Bring the strap up and over the opposite side of the child's instep continuing to the back of the heel and attach to the Velcro® on the bottom of the footplate below the heel. Trim unused portion of strap. Repeat with other footplate.

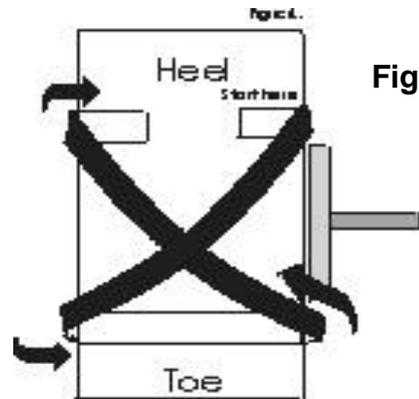


Figure A.

Figure B. Heel Strap

Begin by attaching one end of the shorter (20-inch) black strap close to the bottom of footplate at the toe end. Loop up and back towards the heel. Wrap around heel and down towards the instep strap on the side. Attach firmly to the Velcro® on the bottom. Trim unused portion of strap. Repeat on other footplate.

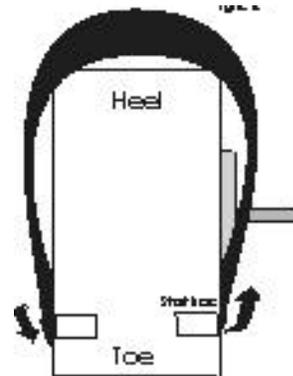


Figure B.

Storage of Straps: Attach loop straps to hook on plates to prevent loss of straps when not in use.

Caution: Never leave child unattended when riding the AmTryke® therapeutic tricycle. Tipping may occur when child tries to dismount with feet strapped to tryke.

For additional accessories for Toddler, see back of this sheet.



The therapeutic tricycle for children with disabilities

For more information contact:
AmTryke®, LLC
PO Box 5127
High Point, NC 27272
1-800-838-1845
www.ambucs.org



Creating Independence For People With Disabilities

Toddler Accessories

These items are available for an additional charge to further adapt the Toddler for your child's use.



Item #30-0115 Toddler Foot Cups
Includes 2 black molded plastic foot cups, 2 14-inch, and 2 19-inch black hook and loop straps.
Cost: \$45.00
Shipping/Handling: Minimum \$10



Item #30-0146 Spacer Blocks
Black Sentra® plastic with black hook and loop for easy attachment. Each block adds an additional more than 1/2-inch in height.
Cost: \$2.75 each
Shipping/Handling: Minimum \$10



Item #8135-000 Stroller Push Bar with Backpack
Attaches to rear deck of tryke.
Cost: \$54.99
Shipping/Handling: Minimum \$10



Item #8136-500 Tandem Insert
Attaches between main frame and rear deck of Toddler tryke allowing a second child to ride. Perfect if you have more than one child.
Cost: \$54.99
Shipping/Handling: Minimum \$10

To place and order contact: AmTryke, LLC at 1-800-838-1845.
Payment Accepted: Visa, MasterCard, and American Express
Please allow 5 to 7 business days for delivery.